i-FIX® Duo

EN	Dual curing adhesive resin cement
LT	Dvigubo kietėjimo dervinis surišimo cementas
DE	Dualhärtender adhäsiver Kunststoffzement
BG	Цимент с двойно втвърдяваща се адхезивна смола
CS	Dvojitě tuhnutí lepicí pryskyřičný cement
DA	Dobbelt hærdende klæbende harpikscement
ET	Kahekordselt kõvenev liimivaigu tsement
FR	Ciment résine adhésif à double durcissement
EL	Τσιμεντό ρητίνης με διπλή σκληρυνση
HU	Kettős kötési ragasztógyanta cement
IT	Cemento resinoso adesivo a doppia polimerizzazione
LV	Divreiz cietējošs līmes sveķu cements
NO	Dobbel herdende limharpiks sement
PL	Klej żywiczny o podwójnym utwardzaniu
PT	Cimento de resina adesiva de cura dupla
RO	Ciment cu răsină adezivă cu întărire duală
SK	Dvojz tvrdnúcí lepiaci živcový cement
SL	Cement iz lepilne smole z dvojnim strjevanjem
ES	Cemento de resina adhesiva de curado dual
SV	Dubbelhärdande limhartscement
TR	Çift kurlenen yapışkan reçine cimentosu
RU	Клей-цемент двойного отверждения
UA	Цемент з клейової смоли подвійного затвердіння

INSTRUCTION FOR USE

DESCRIPTION

i-FIX Duo is a dual curing, bioinert, radiopaque cement, based on methacrylate resins and silica/barium glass filler particles. i-FIX Duo is a high strength, highly resistant device with optimal bonding characteristics. The use of automix tips ensures homogenous mixture which can be placed directly into the restoration.

COMPOSITION

Base: dental glass grinded based on silica and barium 50-70%, methacrylate mixture 30-50%, conditioner 1-5%, silicon dioxide 1-5%, initiator <1%, opacifier <1%, accelerator 1%, inhibitor <1%. Catalyst: dental glass grinded based on silica and barium 50-70%, methacrylate mixture 30-50%, silicon dioxide 1-5%, coinitiator <1%, stabilizer <1%, photoinitiator <1%, inhibitor <1%, pigment <1%.

i-FIX Duo does not contain medicinal substance, including human blood or plasma derivative; tissues or cells, or their derivatives, of human origin; tissues or cells of animal origin, or their derivatives, as referred to in Regulation (EU) No. 722/2012; substances which are carcinogenic, mutagenic, toxic to reproduction or having endocrine-disrupting properties.

PERFORMANCE CHARACTERISTICS

dual curing	light and chemical
flexural strength	115.4±9.4 MPa
water sorption	24.28±0.45 µg/mm ³
water solubility	0.64±0.21 µg/mm ³
film thickness	8.8 µm

INTENDED PURPOSE AND CLINICAL BENEFITS

i-FIX Duo restores/improves aesthetic appearance of restorable tooth; restores/maintains dental function of restorable tooth; protects biological structures of restorable tooth and adjacent tissues.

CLINICAL INDICATIONS

- For the permanent cementation of indirect restorations including ceramic, composite and metal-based inlays, onlays, crowns and bridges;
- For the permanent cementation of posts.

CONTRA-INDICATIONS

Patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients.

RESTRICTIONS TO COMBINATIONS

i-FIX Duo should not be used with products containing eugenol, hydrogen peroxide as they may disturb polymerization process.

UNDESIRABLE SIDE EFFECTS

In susceptible individuals, i-FIX Duo may cause allergic or irritation reactions (skin, eye, mucosa, respiratory tract).

RESIDUAL RISKS

Risk control measures have been implemented and verified, risk is reduced as far as possible, the overall residual risk is judged to be acceptable.

PATIENT TARGET GROUP

No restrictions known regarding patient population, their age and general health conditions. There may be children, middle aged or elderly patient.

INTENDED PART OF THE BODY OR TYPES OF TISSUES OF BODY FLUIDS

Part of the body – mouth. Tissues or body fluids contacted by the device – tooth.

INTENDED USER

i-FIX Duo is developed for professional use in dentistry only. Its user only licensed doctor who has knowledge how to use common dental cements. There is no need for specific training.

STERILITY

i-FIX Duo is delivered non-sterile. There is no need of any preparatory sterilization, cleaning or disinfection, preventive, regular maintenance or calibration to ensure that the device operates properly and safely during its intended lifetime. However, do not use if primary package is damaged.

USE ENVIRONMENT

i-FIX Duo is designed to be used in dental office where ambient temperature is 18-25°C. Dispensed amount of cement is suitable for single use (only for one patient). Do not re-use. Dispensed amount kept not in original package may lead to loss of function.

CONSUMABLE COMPONENTS AND ACCESSORIES

No accessories are supplied with the device. Consumables, such as automix tips, intraoral-root canal tips are supplied with the device.

INSTRUCTION FOR USE

PREPARATION:

1. Remove the temporary restoration.
2. Clean the surface, rinse with water and dry with air. Avoid over drying. The surface must remain glossy.
3. Try in the permanent restoration to ensure proper fit.
4. Apply layer of etch, such as i-GEL^N on all surface to be etched, follow manufacturer's instruction for use. Leave etch in place for 15 seconds (dentine), 30 seconds (enamel). Rinse with water and dry with air. Avoid over drying dentin. After drying, the surface appears chalky and must not be contaminated prior to applying the adhesive. If the surface becomes contaminated with saliva, rinse and dry again and re-etch if necessary.
5. Apply a thin layer of light curing adhesive, such as i-BONDING LC^N immediately onto etched surface or dual cure adhesive for post cementation, follow manufacturer's instruction for use.
6. Remove i-FIX Duo syringe cap.
7. Promptly and carefully attach the automix tip to the syringe.
8. Test flow of material from automix tip before using.
9. Squeeze out a 1-2mm of paste onto a mixing pad to equal the base paste and catalyst paste in the automix tip. Do not use the paste which has been squeezed out.

PLACEMENT:

INLAYS/ONLAYS:

1. Dispense the cement directly into the restoration. Cover all surfaces.
2. Seat the restoration firmly onto the preparation. Stabilize until cement is fully set.

CROWNS/BRIDGES:

1. Dispense the cement directly into the restoration. Cover all surfaces.
2. Seat the restoration firmly onto the preparation. Let the cement flow from all margins. Stabilize until cement is fully set.

POSTS:

1. Mount the intraoral-root canal tip onto the automix tip and dispense the cement directly into the canal.
2. Seat the post carefully and rotate slightly to avoid trapping air. Let the cement flow from the canal space. Light cure for 10 seconds to fix the position of the post. Use LED polymerization lamp with light intensity 1200mW/cm². Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use. Stabilize until cement is fully set.

FINISHING:

1. After the restoration is properly seated and while the cement is in rubbery state remove all excess cement with scaler or explorer. Rubbery state can be achieved by light cure for approx. 2-3 seconds or by allowing the cement to self-cure for approx. 30 seconds - 1 minute after application.
2. Light cure for 20-30 seconds all surfaces and margins after removal of all excess cement. Use LED polymerization lamp with light intensity 1200mW/cm². Some lamps with higher intensity could require less time of polymerization, follow manufacturer's instruction for use. Finish restoration.

NOTE!

- a) Working time at 23°C is about 1 minute 30 seconds to 2 minutes. Chemical setting time at 37°C is about 2 minutes to 2 minutes 30 seconds. Light curing time per surface is 20-30 seconds.

WARNINGS

Tips are single use only. Do not use in direct contact with pulp tissue. As i-FIX Duo highly adhesive, clean all instruments before cement set on. The setting time depends on the ambient and oral temperature. Increased temperature will reduce the working/setting time. Vice versa decreased temperature will prolong the working/setting time. The cement is sensitive to light. The working time significantly reduces during application under intensive lighting. When removing i-FIX Duo from refrigerator after storage, leave the material at ambient temperature for some time until warm-up. After the desired amount of material extruded, immediately close the syringe, so that the material is not unlighted. Do not use i-FIX Duo for patients who have a history of severe allergic or irritation reactions to product or to any of the ingredients. i-FIX Duo does not emit radiation and does not cause any electromagnetic interferences.

PRECAUTIONS!

It is recommended to use cofferdam during application of the product. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF SWALLOWED: Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell. IF INHALED: Remove person to fresh air and keep comfortable for breathing. Wash hands thoroughly after handling. Use only in a well-ventilated area. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

Precautions to be taken in the event of changes in the performance of the device:

If, during the use of the product, noticed non-homogenous consistency, product does not harden or does not adhere on the applied surface while curing with light or any by-products/phases are released during curing, or sudden acute pain occur on application site, or if any other abnormal behavior of the product noticed while manipulating the device, that is not mentioned above, discontinue to use immediately. Remove the restoration from the tooth cavity with suitable dental instrument do not let the product to be swallowed. Ask patient how she/he is feeling. If patient noticed any undesirable side-effects, immediately call to a local poison center. Collect all available remaining supplies, do not use them again and keep them out of reach in a safe place until further notice. Contact the manufacturer immediately and report of any noticed changes in the performance of the product.

SHELF-LIFE

Shelf-life of i-FIX Duo is 2 years from the date of manufacture. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date.

STORAGE

Keep product tightly closed in dry well-ventilated place at 2-8°C (preferable in refrigerator). Protect from direct sunlight and heat sources. Do not freeze. Keep out of the reach of children!

DISPOSAL

Dispose of contents/container to as required by national regulatory requirements.

VIGILANCE

If any serious incident that has occurred in relation to the device report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

i-FIX Duo is safe and performs as intended if it is used in accordance to manufacturer's instruction for use. Summary of safety and clinical performance is available through manufacturer's website www.i-dental.lt/sscp/ until European Database on Medical Devices (EUDAMED) comes online.

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

VALIDITY

Upon publication of this instruction for use, all previous versions are superseded.

PACKAGING

REF	IFDTP	8g double syringe, 10pcs automic tips (for inlays/onlays, crowns, bridges), 10pcs intraoral-root canal tips (for posts)
-----	-------	---

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