# Luxatemp Star

## Inhalt | Contents

Gebrauchsinformation-DEUTSCH	4
Instructions for use-ENGLISH	13
Mode d'emploi-FRANÇAIS	22
Istruzioni d'uso-ITALIANO	32
Instrucciones de empleo-ESPAÑOL	42
Instruções de uso-PORTUGUÊS	52
Gebruiksaanwijzing-NEDERLANDS	62
Brugsanvisning-DANSK	72
Användarinstruktioner-SVENSKA	81
Instrukcja użycia-POLSKI	90
Информация по применению-РУССКИЙ	100
<b>使用说明-简体中文</b>	<b>110</b>
<b>사용 설명서-독일어</b>	<b>118</b>
使用説明書-日本語	127

## Produktbeschreibung

---

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## Product description

---

Luxatemp Star is a self-curing composite for the fabrication of temporary crowns and bridges, inlays, onlays and veneers. The 2-component material is based on multifunctional methacrylates and is automatically mixed.

## Indications

---

- Fabrication of temporary crowns, bridges, inlays, onlays, partial crowns and veneers.
- Fabrication of long-term temporaries.

## Contra-indications

---

Do not use the material in cases of allergies to any of the substances contained or, as can occur in rare cases, in the event of contact allergy.

## Basic safety instructions

---

- For dental use only.
- Keep away from children.
- Avoid contact with skin and eyes. In the event of inadvertent contact, rinse thoroughly with plenty of water immediately and consult a physician if necessary.
- Wear face mask and goggles when finishing the temporary, otherwise sanding dust may get into the eyes and respiratory tract.

## Application system

---

- Automix cartridge: see “Using the Automix cartridge” on page 17
- Smartmix syringe see “Using the Smartmix syringe” on page 18

**Note:** When using a cartridge or syringe for the first time, express a pea-sized amount of material and discard it.

## Timing

---

0:00–0:45 min	Insertion in the mouth
1:30–2:30 min	Removal from the mouth
5:00 min	End of setting Finishing of the temporary

**Note:** Times refer to normal room temperature (23 °C/72 °F) and a normal relative humidity of

air (50 %). Higher temperatures shorten, lower temperatures increase those times.

## Recommended use

---

1. Before preparation of crowns or bridges, take an impression using alginate, silicone or a thermoplastic impression material. Alternatively, the temporary can be made using a laboratory-prepared thermoforming matrix or, in the case of single crowns, with a strip crown. Strictly follow the appropriate manufacturer’s instructions.

**Note:** Using a silicone impression, undercuts have to be blocked out and if necessary pressure relief channels may have to be cut in. To create a connection with the abutment teeth in the case of a gap in the molar region, cut a channel into the impression.

2. Dry the prepared teeth and block out any undercuts in the preparation. Then lightly lubricate the prepared teeth and surrounding tissue plus any composite core build-ups using Vaseline or a similar separating medium.
3. Match shade and place appropriate Luxatemp Star cartridge into the Automix-Dispenser or use appropriate Smartmix syringe (see “Using the Automix cartridge” on page 17/“Using the Smartmix syringe” on page 18).

**Note:** To avoid bubbles, it is important that the end of the mixing tip always remains immersed in the material during application.

When dispensing the material, keep excess to a minimum.

4. Fill the impression or the thermoforming matrix with Luxatemp Star. Dispense the material onto the occlusal surface of the

impression or thermoforming matrix first then fill to the gingival margin.

5. Insert the impression or thermoforming matrix, filled with Luxatemp Star, over the prepared teeth within 45 seconds after start of mixing, using moderate pressure and fix in place.

**Note:** The setting reaction has to be monitored intra-orally (e.g. using a scaler), because the curing is significantly affected by the temperature in the mouth and the temporary can only be removed during its elastic phase.

6. The temporary must be removed from the mouth during the elastic phase, i.e. during the period from 1:30 to 2:30 minutes after start of mixing.

**Note:** To improve the precision of large temporaries, they can be put back into the impression after removal from the mouth until reaching its final hardness.

7. Before finishing, remove the oxygen inhibition layer from the surface of the temporary (e.g. with alcohol).

**Caution!** When finishing the temporary, sanding dust may get into the eyes and respiratory tract.

- ▶ Wear face mask and goggles.
- ▶ Do not inhale the sanded dust and use an evacuator.

8. After about 5 minutes from the start of mixing, the temporary can be finished using cross-toothed grinders, flexible disks, etc.
9. Check occlusion and grind if necessary using suitable instruments.
10. If necessary, the surface of the temporary may be glazed with a light-curing high-gloss varnish (e.g. Luxatemp-Glaze & Bond) or polished manually with a rubber polisher or goat's hair wheel.
11. Clean the prepared tooth, dry it and thoroughly remove the separating medium.

12. Cement the temporary.

**Note:** Any temporary cement can be used for cementing the temporary. DMG recommends using automatically mixing luting cements, e.g. TempoCemNE (non-eugenol zinc oxide cement).

Cements containing eugenol can inhibit the polymerization of resin-based luting cements for the permanent cementation.

13. Check occlusion and adjust accordingly.

## Customizing and repair of temporaries

---

**Note:** When customizing and repairing a freshly-made temporary it does not need to be roughened and a bonding agent is not required (proceed with step 2).

**Caution!** When working on the temporary, sanding dust may get into the eyes and respiratory tract.

- ▶ Wear face mask and goggles.
- ▶ Do not inhale the sanded dust and use an evacuator.


1. Repair of an older temporary: Roughen the temporary slightly using a grinder or a sandblaster at the appropriate points. Then apply a bonding agent (e.g. Luxatemp-Glaze & Bond) according to manufacturer's instructions.
2. Apply Luxatemp Star to all surfaces.
3. Hold the parts together for 3 minutes.
4. After 5 minutes, the appropriate points can be finished as usual.

**Note:** To speed up the polymerization, the finished temporary can be placed in warm water at 50 °C (120 °F) for a few minutes.

Alternatively, the temporary can also be customized or repaired using the light-curing flowable

composite LuxaFlow Star. Follow the associated instructions for use.

## Using the Automix cartridge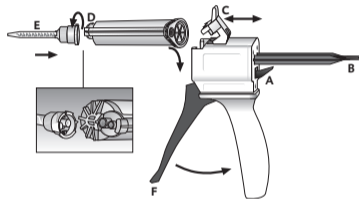

### Insert cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the cartridge and lock it by pushing down the plastic lever [C].

**Note:** Make sure that the notches on the cartridge line up with those on the Automix-Dispenser.

### Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

**Note:** Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it into place.

### Apply material

- ▶ When using a cartridge for the first time, express a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [F] of the Automix-Dispenser and it is then applied directly.


**Note:** Leave the used mixing tip on the cartridge after use. It serves as a cap.

### Remove cartridge


1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

### Using the Smartmix syringe

---

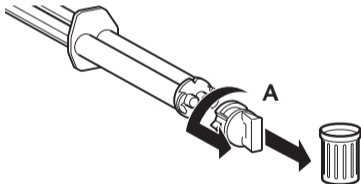
1. Before attaching the mixing tip, remove the cap [A] or used mixing tip after turning it 90° counterclockwise, and discard it.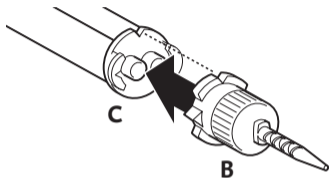

2. Attach a new mixing tip [B].

**Note:** Make sure that the notches on the mixing tip line up with those on the Smartmix syringe [C].

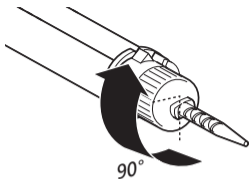3. Turn the mixing tip clockwise 90° to lock it into place.
4. The material is mixed in the mixing tip when it is expressed and can then be applied directly.

**Note:** When using a syringe for the first time, express a pea-sized amount of material and discard it.

Leave the used mixing tip on the Smartmix syringe after use. It serves as a cap.

## **Disinfection of the Automix-Dispenser**

The Automix-Dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix-Dispenser can also be autoclaved.

## **Side effects**

There are no known systemic side effects to date.

## **Interactions**

Surpluses of material on methacrylate basis can change the setting properties of silicone impression and bite registration materials.

- ▶ If a restoration has been made previously, clean the preparation and surrounding tissue

carefully with gauze soaked in alcohol and water spray before taking the impression.

## **Composition**

Glass filler in a matrix of multifunctional methacrylates; catalysts, stabilizers and additives. Free of methyl methacrylate. Total filler volume: 44 w. % = 24 vol. % (0.02 to 1.5  $\mu\text{m}$ )

## **Storage and shelf life**

- ▶ Store in a dry place at room temperature (15 to 25 °C/59 to 77 °F).
- ▶ Protect against exposure to direct sunlight.
- ▶ Leave the used mixing tip on the Automix cartridge after use. It serves as a cap.
- ▶ Leave the used mixing tip on the Smartmix syringe after use. It serves as a cap.
- ▶ Do not use after the expiration date!

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**DMG** Chemisch-Pharmazeutische Fabrik GmbH