



CLEANING AND DISINFECTION PRODUCT FOR MEDICAL DEVICES IN DENTAL ENVIRONMENTS

DENTASEPT® SH PRO Wipes

DENTASEPT SH PRO WIPES ARE A READY-TO-USE CLEANING AND DISINFECTION PRODUCT FOR MEDICAL DEVICES IN A DENTAL ENVIRONMENT

They combine broad spectrum efficacy, short contact times, excellent material compatibility and the properties of a gentle cleaner.

Dentasept SH Pro Wipes are suitable for use on a wide range of devices and can be used in practice as intended in your protocols, delivering the disinfection results and compliance level needed to restrict the spread of pathogens.

- Can be used on alcohol sensitive devices
- Efficient formula without perfume and dye
- Cleaning and disinfection in one step, leaving the device visibly clean
- 100% Biodegradable viscose wipes



DENTASEPT® SH PRO Wipes

Combining broad spectrum efficacy, short contact times, excellent material compatibility (including alcohol sensitive objects) and the properties of a gentle cleaner

PRODUCT CHARACTERISTICS

- Cleans and disinfects in one step
- Without alcohol, perfume and dye
- Broad spectrum antimicrobial and short contact time
- pH 6
- Formula ensures compatibility with a wide range of materials, including polymers (e.g. polycarbonate, plexiglas)
- Dentasept SH Pro Wipes substrate is 100% viscose

INSTRUCTIONS FOR USE



1 Ready-to-use wipes. Open shelf life based on EN 16615: 3 months.



2 Apply to the area of the medical device to be treated, using as many wipes as necessary.



3 Minimum contact time: 2 minutes. Thoroughly rinse with tap water those treated areas of the medical device intended to come into contact with skin or oral mucosa, except for products treated by wrapping.



4 Close the packaging properly after each time it is opened. Do not reuse wipe. Disinfection can no longer be guaranteed.

MICROBIOLOGICAL PROPERTIES

CLAIM/CONDITIONS	CONTACT TIME
Bactericidal* - According to: EN 16615 (2 min), EN 13727 (1 min)	2 min
Yeasticidal* - According to: EN 16615 (2 min), EN 13624 (2 min)	2 min
Tuberculocidal* - According to EN 14348	60 min
Active against enveloped viruses* - According to EN 14476	1 min
Active against Polyomavirus SV 40* - According to EN 14476	5 min
Active against Rotavirus* - According to EN 14476	30 sec
Limited spectrum virucidal* - According to EN 14476	30 min
Adenovirus	30 min
Norovirus (MNV)	15 min

* Dirty conditions

ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
Dentasept SH Pro Wipes	6 x 100 wipes (180 x 200 mm)	XXXXXX



D.M.D.
1, Rue de l'Espoir
59260 Lezennes
France

Distributed by:
LABORATOIRES ANIOS
1, Rue de l'Espoir
59260 Lezennes
France

APPLICATION AREA

Medical device treatment in dental environment:

- Between each patient: cleaning and disinfection of non immersible and non-invasive medical devices, such as curing lamp bodies;
- Before the step of cleaning, disinfection and sterilisation of medical devices (for example, rotary hand pieces, turbines, contra-angles), and in compliance with the indication of the manufacturer: wrapping in case of transport to the sterilisation room and/or wiping.

INSTRUCTIONS FOR USE

- 1) Ready-to-use wipes. Open shelf life based on EN 16615: 3 months.
- 2) Apply to the area of the medical device to be treated, using as many wipes as necessary.
- 3) Minimum contact time: 2 minutes. Thoroughly rinse with tap water those treated areas of the medical device intended to come into contact with skin or oral mucosa, except for products treated by wrapping.
- 4) Close the packaging properly after each time it is opened. Do not reuse wipe. Disinfection can no longer be guaranteed.

INGREDIENTS

Impregnation solution: Didecyl-dimethylammonium chloride, excipients.

CERTIFICATES

Medical Device. Full antimicrobial activity for Dentasept SH Pro Wipes is included in the scientific dossier, available on request.

STORAGE

Storage: between +5°C and +25°C.

MEDICAL DEVICE USE SAFELY

Please read label and product information.



FOR PROFESSIONAL USE ONLY.

