## Canasol (EN)

**Description:** Canasol is an aqueous sodium hypochlorite solution. At low concentrations dissolve mainly necrotic tissue, at higher concentrations also dissolves vital tissue.

**Procedure:** It is used for the irrigation and debridement of root canals during endodontic treatment. Remove smear layer with appropriate irrigant such as EDTA solution or Citric Acid 12,5%. Only for professional use.

### Instructions for use:

1. Remove the seal.
2. Fill syringe with Canasol (10 ml recommended).
3. Attach irrigation tip securely to syringe.
4. Irrigate as desired.
5. Remove and dispose of syringe and irrigating tip.

### Precautions:

1. Read MSDS for material handling and safety information.
2. Irrigation should not be irritant for the apical area.
3. Avoid spilling on surrounding tissues.
4. Avoid spilling on colored fabrics and surfaces as Canasol is moderately alkaline and has a whitening effect.

### Warnings:

1. Caution, caustic!
2. Keep out of the reach of children.
3. Avoid contact with skin and eyes.
4. Do not swallow or inject.
5. Avoid mixing Canasol with CHX.

**Storage:** Store in a dry area away from heat and direct sunlight.

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