



### General Tips for Your Success

- Always wear gloves; use of rubber dam is recommended
- Always put protective silicon cover over contra-angle for simultaneous length control
- Always insert lip clip into the cheek opposite the tooth being treated
- Always dry access cavity with air blower or cotton ball
- Separate length determination: choose suitable hand file size (matching the root canal diameter)
- First determine working length using a hand file, then control length simultaneously via the contra-angle during preparation

### 1. Activating the Apex Locator




Touch micromotor with lip clip  
Display will show: APEX LOCATOR ON

### 2. Separate Length Determination




Clamp hand instrument into file clip and hang lip clip into patient's cheek pouch

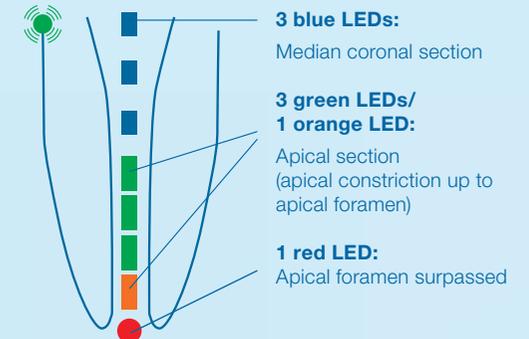

→ The working length is set at the 3rd green LED

### 3. Simultaneous Length Control

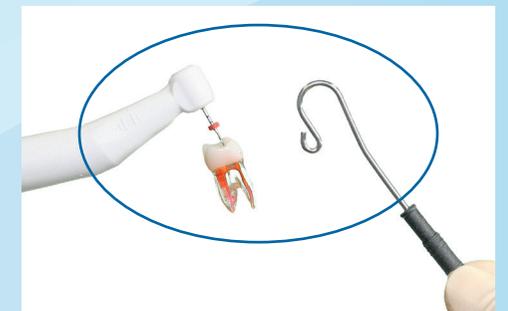

Insert rotary or reciprocating file into contra-angle and hang lip clip into patient's cheek pouch

4. Option: APICAL-AUTO STOP can be activated via the menu



### 1. Lubricating the 6:1 Contra-Angle (With the help of compressed air)

① AIR 5 sec. MON TUE WED THU FRI SAT SUN

② OIL 1 sec. MON TUE WED THU FRI SAT SUN

③ OIL 1 sec. MON TUE WED THU FRI SAT SUN

④

⑤

⑥ CAL

### 2. Surface Disinfection

① ✓ Aldehyde-free solution

② ✓

DO NOT spray display directly!

### 3. Charging the Battery

✓ Symbol flashing red: Charge

Charging time approx. 3 h

Consider the following points before charging:

- Battery symbol already changes from red to green after **approx. 30 min.**
- At this point, the battery has a capacity of only 20 %.
- Do not stop charging; charge battery for 3 full h.

After battery is fully recharged, a continuous use of 2 h is possible.

Symbol flashing green: **DO NOT** charge