BIOFILM **DISCLOSER**

INSTRUCTIONS FOR USE

**"I FEEL
GOOD"**

EMS⁺

SUMMARY

EN	ENGLISH	p.2
DE	DEUTSCH	p.2
FR	FRANÇAIS	p.2
IT	ITALIANO	p.2
ES	ESPAÑOL	p.3
PT	PORTUGUÊS	p.3
NO	NORSK	p.3
SE	SVENSKA	p.3
FI	SUOMI	p.4
NL	NEDERLANDS	p.4
DK	DANSK	p.4
PL	POLSKI	p.4
CZ	ČESKÝ	p.5
SI	SLOVENŠČINA	p.5

BG	БЪЛГАРИЯ	p.5
HU	MAGYAR	p.5
HR	HRVATSKA	p.6
RO	ROMANIAN	p.6
SK	SLOVENSKY	p.6
GR	ΕΛΛΗΝΙΚΑ	p.6
ET	EESTI KEEL	p.7
LV	LATVIEŠU	p.7
LT	LIETUVIŠKAI	p.7
RU	РУССКИЙ	p.7

PRE-SOAKED PELLETS FOR 2 TONE BIOFILM DISCLOSURE

INTENDED USE

Pre-soaked pellets for 2 tone biofilm disclosure. Red for new biofilm - Blue for mature biofilm.

INTENDED USER

Only qualified dentists, dental hygienists and dental professionals must use this product.

PATIENT POPULATION

Biofilm discloser pellets are intended for use on patients who are undergoing professional dental prophylaxis treatment.

TREATMENT RECOMMENDATIONS

Use a pair of tweezers to apply the solution contained in the pellet onto all supragingival surfaces with light pressure. Use one pellet per treatment. Discard the pellet after applying the solution and rinse. Start removing the biofilm with AIRFLOW® after rinsing.

GENERAL PRECAUTIONS


1. Avoid contact with eyes. In case of contact, immediately flush with water and seek medical attention.
2. Avoid contact with skin, clothing or equipment. In case of contact, immediately wash with water.
3. Avoid contact with dental cavity. If case of contact, immediately wash with water. And if the area are stained, the color will be gradually removed.
4. This product contains red and blue dyes, which will stain dental restorative materials.
5. Do not swallow.
6. Do not mix with other products.
7. Keep out of reach of children.

COMPONENTS / INGREDIENTS

Aqua, Glycerin, CI 45430, CI 42051, Ethylparaben, Aroma

CONTENT:

250 pellets

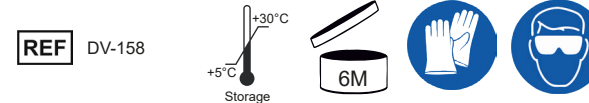Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

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Dr. Wittmann GmbH & Co KG
E.M.S. Electro Medical Systems S.A.