## GC KALORE

Prior to use, carefully read the instructions for use. **EN**

LIGHT-CURED RADIOPAQUE UNIVERSAL COMPOSITE RESTORATIVE

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS



- Direct restorative for Class I, II, III, IV, V cavities.
- Direct restorative for wedge-shaped defects and root surface cavities.
- Direct restorative for veneers and diastema closure.

### CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE


- Shade Selection
  - Clean the tooth with pumice and water. Shade selection should be made prior to isolation. Select the appropriate shades by referring to the GC KALORE Shade Guide or Multi Shade Build-Up Guide.
- Cavity Preparation
  - Prepare cavity using standard techniques. Dry by gently blowing with oil free air. Note: For pulp capping, use calcium hydroxide.
- Bonding Treatment
  - For bonding GC KALORE to enamel and / or dentin, use a light-cure bonding system such as GC Fuji Bond L.C, GC UniFil® Bond or GC G-BOND (Fig. 1).
- Placement of GC KALORE
  - Dispensing from a Unilit
    - Insert the GC KALORE Unilit into a commercially available applicator (Centrix Applicator is recommended). Refer to the applicator manufacturer's instructions for use. Remove the cap and extrude material directly into the prepared cavity. Use steady pressure (Fig.2).
  - Dispensing from a syringe
    - Reseal the syringe cap and dispense material onto a mixing pad. Place the material into the cavity using a suitable placement instrument. After dispensing, screw syringe plunger anticlockwise by a half to full turn to release residual pressure inside the syringe. Replace cap immediately after use. Note:
      - Material can be applied in a single shade layer to achieve aesthetic restorations using Universal shades. For details, refer to the Clinical Hints.
      - Material may be hard to extrude immediately after removing from cold storage. Prior to use, leave to stand for a few minutes at normal room temperature.
      - After dispensing, minimise exposure to ambient light. Ambient light can shorten the manipulation time.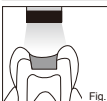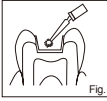

### Clinical Hints

- In the case of small cavities
  - Restore using a one shade layer technique. In most cases the use of one Universal shade alone will be sufficient. In cases where a higher degree of translucency is needed, one of the Translucent shades can be selected.
- In the case of large and/or deep cavities
  - In most cases a multi shade layering technique will give the best aesthetic results. To block out shine through from the oral cavity or to mask discoloured dentine, select an appropriate Opaque shade and continue to build up with a Universal shade. For optimal aesthetics use a Translucent shade as the final composite layer. In the case of deep posterior cavities, a flowable composite such as GC GRADIA DIRECT Flo /LoFo or a glass ionomer cement such as GC UNIBOND L.C (Paste Pak) or GC Fuji IX GP can be used on the cavity floor instead of an Opaque shade. See also Examples of Clinical Applications and/or Shade Combination Chart.

### Examples of Clinical Applications (Clinical Hint)

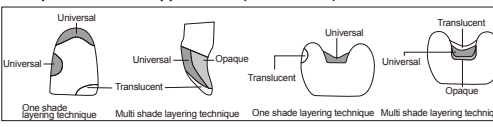OPACQUE	A1	A2	A3	A3.5	A4	B1	B2	B3	C2	C3	D2	D3	CVD	BW	XBW
Opacque	OBW	AO2	AO3	AO3	AO4	OBW	AO2	AO3	AO3	AO4	AO4	AO4	AO4	OBW	OBW
Universal	A1	A2	A3	A3.5	A4	B1	B2	B3	C2	C3	D2	D3	CVD	BW	XBW
Translucent (Enamele)	WT	WT	DT	DT	DT	WT	WT	DT	DT	DT	DT	DT	DT	DT	WT

- Contouring before Light Curing
  - Contour using standard techniques.
- Light Curing
  - Light cure GC KALORE using a light curing unit (Fig. 3). Keep light guide as close as possible to the surface. Refer to the following chart for Irradiation Time and Effective Depth of Cure.

GC KALORE - Irradiation Time and Effective Depth of Cure	Irradiation time	Plasma arc (2000mW/cm <sup>2</sup> )	3 sec.	6 sec.
CT, NT, WT, GT, CVT	GC G-Light (1200mW/cm <sup>2</sup> )	10 sec.	20 sec.	20 sec.
A1, A2, B1, B2, C2, D2, XBW, BW, DT	Halogen / LED (700mW/cm <sup>2</sup> )	20 sec.	40 sec.	40 sec.

- Finishing and Polishing
  - Finish and polish using diamond burrs, polishing points and discs. To obtain a high gloss, polishing pastes can be used.

### SHADES

- 26 shades
- XBW (Extra Bleaching White), BW (Bleaching White), A1, A2, A3, A3.5, A4, B1, B2, B3, C2, C3, D2, CV (B5:Cervical), CVD (B7:Cervical Dark)
  - 5 Opaque shades (color code on unilit cap / syringe label: violet) AO2, AO3, AO4, OBW (Opaque Bleaching White), OXBW (Opaque Extra Bleaching White)
  - 6 Translucent shades (color code on unilit cap / syringe label: gray) WT (White translucent), DT (Dark translucent), CT (Clear translucent), NT (Natural translucent), GT (Gray translucent), CVT (Cervical translucent)
- A, B, C, D shades are based on Vita® \*\* Shade.
\*\*Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

### STORAGE

Store in a cool and dark place (4-25°C / 39.2-77.0°F) away from high temperatures or direct sunlight. (Shelf life : 3 years from date of manufacture)

### PACKAGES

- Unipits
  - Refill
    - a. Pack of 20 tips (each in 11 shades) (0.16mL per tip) A1, A2, A3, A3.5, A4, B1, B2, B3, C2, C3, D2
    - b. Pack of 10 tips (each in 15 shades) (0.16mL per tip) XBW, BW, CV, CVD, AO2, AO3, AO4, OBW, OXBW, WT, DT, CT, NT, GT, CVT
    - Note : Weight per Unilit<sup>®</sup> : 0,3g
  - Option
    - a. Shade guide b. Mixing pad (No.14B)
- Syringes
  - Refill
    - 1 syringe (in 26 shades) (2.0mL per syringe)
    - Note : Weight per syringe : 4g
  - Option
    - a. Shade guide b. Mixing pad (No.14B)

### CAUTION

- In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take care to avoid ingestion of the material.
- Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
- For infection control reasons, Unipits are for single use only.
- Wear protective eye glasses during light curing.
- When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
- Do not mix with other similar products.
- Avoid getting material on clothing.
- In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge, or cotton pellet before light curing.
- Do not use GC KALORE in combination with eugenol containing materials as eugenol may hinder GC KALORE from setting.

Last revised: 05/2009

**0086**

**GC**

MANUFACTURED BY  
GC DENTAL PRODUCTS CORP.  
2-285 Torimitsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED BY  
GC CORPORATION  
76-1 Hasunuma-cho, Habashi-ku, Tokyo 174-8585, Japan

**EU: GC EUROPE N.V.**  
Researchpark Hassrode-Leuven 1240, Interleuvenlan 33,  
B-3001 Leuven, Belgium TEL.: +32 16 74 10 00

**GC AMERICA INC.**  
3727 West 123th Street, Alsip, IL 60803 U.S.A.  
TEL.: +1 708-597-0900

**GC ASIA DENTAL PTE. LTD.**  
11 Loyang Way, #05-27 Singapore 508724  
TEL.: +65 6546 7588

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