Opalescence™ Endo

Description: Opalescence Endo is a 35% hydrogen peroxide gel specially formulated for the medical "walking" whitening technique.

Indications:

Use Opalescence Endo to whiten endodontically treated, internal discolored teeth caused by disease, injury, or medical treatment.

Procedure:

- Verify healthy periodontal tissues and quality endodontic treatment.
- Establish the expectations of the patient. Explain any esthetic limits before proceeding.
- Completely remove all the restorative and root sealing material from the coronal pulp chamber and 2-3mm below healthy gingiva. Place a 2mm thick conventional glass ionomer or a resin modified glass ionomer to seal the endodontically treated canal. Verify set of material before proceeding.
- Attach Black Mini™ tip or Micro 20 ga tip to syringe and express Opalescence Endo into the coronal pulp chamber avoiding soft tissues. Place cotton pellet over gel. Make sure no whitening agent is on margins of access to ensure temporary seal.
- Leave 1.0 to 1.5mm of space to accommodate the provisional restoration. Do not place Opalescence Endo on oral tissues. Should contact occur, rinse immediately with lots of water.
- Seal access with a self-curing, non-eugenol, and non-resin provisional restoration such as UltraTemp™ Regular. Verify occlusal clearance in protrusive and lateral movements. Premature occlusal contact may fracture the provisional restoration resulting in leakage and loss of whitening material, thus prolonging treatment.
- Repeat the procedure every 1 to 5 days until the desired color change is achieved or no further benefit is noticed.
- Remove provisional restorative material and whitening material to the level of glass ionomer sealing material. Rinse and clean access opening. Fill with cotton pellet and temporary restoration material. Restore with adhesive dentistry procedures.
- Wait 7-10 days following whitening procedures before placing restorations as whitening materials interfere with bonding agents.

Precautions:

- Over years, regression is to be expected and re-whitening on an average of 3-5 years is recommended.
- All Ultradrant syringe-delivered dental materials should not be injected.
- For dentist storage, material should be refrigerated to maintain shelf life.
- Dispose of tips and syringe according to local regulations
- Pregnant or breastfeeding women, or patients with serious health concerns should not whiten. Consult their primary care provider prior to treatment.
- Do not use on patients with a known sensitivity to peroxides, glycols, acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.
- Keep Opalescence Endo off soft tissues.
- Do not express so much gel that there is a risk of ingestion
- Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check. If resistance continues, DO NOT use that syringe and contact Ultradrant Customer Service for replacement.
- To avoid cross-contamination, use a disposable syringe cover, re-cap, and wipe syringe with an intermediate disinfectant between uses. If these measures are not taken, the syringes should be considered single-use.

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